TB Specimen Packing and Shipping Instructions

Shipping Components:

- 1 Laboratory Test Request Form
- 1 Sterile Specimen collection tube
- 1 Absorbent sheet
- 1 Primary specimen bag (small biohazard bag)
- 1 Secondary specimen bag (large biohazard bag)

Packing Instructions:

- 1. Properly and clearly label the specimen tube with the patients name, DOB, and date and time of collection.
- 2. Following specimen collection guidelines, collect sputum in the provide specimen tube.
- 3. After collection, tighten the tube as to avoid leakage.
- Place the specimen tube inside the primary specimen bag containing the absorbent sheet material (small biohazard bag) and seal the bag.
- Place that primary specimen container inside the secondary specimen bag (large biohazard bag) and seal the bag.
- 6. Complete the required information on the provided laboratory test request form.
- The completed laboratory test request form should be folded and placed inside the outside pocket of the secondary specimen bag.



Pictured above: absorbent sheet, specimen tube, primary and secondary specimen bags Pictured below: Laboratory Test Request Form and completed packaging.



